
TC103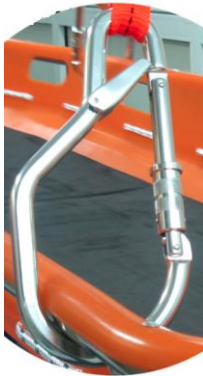

EN	STRETCHER LIFTING SLING	OPERATING INSTRUCTIONS
FR	CIVIERE ELINGUE	NOTICE D'UTILISATION ETD'ENTRETIEN

		
EN	The four point lifting sling consists of four karabiners, attached to a sling which has a twist-gate carabiner attached to the other end. The four karabiners lock onto each corner of the stretcher to enable it to be lifted.	Adjustment of lengths if required can be done by with the help of the buckle provided on the product
FR	L'élingue à quatre points se compose de quatre mousquetons, attachés à une fronde qui a une torsion porte-mousqueton attaché à l'autre extrémité. Les quatre mousquetons verrouillés sur chaque coin de la civière pour lui permettre d'être soulevé.	Le réglage de la longueur, si nécessaire, peut être effectué à l'aide de la boucle prévue sur le produit.

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EN
OPERATING INSTRUCTIONS

WARNING!

This notice must be translated (eventually), by the retailer into the language of the country of use. For your safety, comply strictly with the instruction for use, verification, maintenance and storage.

DELTAPLUS cannot be held liable for any direct or indirect accident occurring as a result of use other than provided for in this notice. Do not use this equipment beyond its capabilities!

DESCRIPTION:

The Stretcher lifting sling has been designed for rescue operation and is manufactured from raw material conforming to the requirements of fall arrest standards. It is specially designed for vertical transport of stretchers (TC102). It is constructed of four polyester webbing lanyards(EN354) with auto lock karabiners for lifting the stretcher. The lanyards are connected to two length adjusting webbing legs to maintain the stretcher in the horizontal position. It is simple to use for various application (mountaineering, industrial rescue and helicopter winching). This product can be used with TC102 stretchers. The four karabiners lock onto each corner of the stretcher to enable it to be lifted or lowered horizontally. The assembly has a safe working load limit of 280 kg.

LABELLING FOR STRETCHER LIFTING SLING: You will find on the product:

- the trademark: DELTAPLUS®
- the product reference, for example TC103
- the batch number, example A132228
- indication of conformity with the directive (CE)
- Instruction of use

Product markings should be checked periodically for legibility

USE AND PRECAUTIONS:

The stretcher lifting sling should be connected to the stretcher by using the auto lock(AM017) karabiners (EN362). The level of the stretcher should be maintain horizontal by adjusting the blue straps through the adjusting buckle.

Strap the patient properly to the stretcher before attempting any lift.

Do a trial lift to check the level of the stretcher before attempting the final lift.

Suitable for use of the stretcher (TC102)

- Before and during use, we recommend that you take every precaution necessary for an eventual safe rescue.
- Check the state of the before each use: visual inspection of the straps (no sign of cuts, burns or abnormal shrinkage), the seams (no visible damage), the metallic parts (no deformation or rust) and that the connectors are working properly.

The stretcher lifting sling should not be re-used in the event of deformation, damage or doubt.

It is forbidden to add or replace any component whatsoever.

TECHNICAL DATA:

Strap material:polyester (27mm wide Lanyard conforming to EN :354), Karabiners (EN362), and high quality steel fittings.

USAGE LIMITS:

This products has a maximum life of 10 years in storage (from the date of manufacture), or 7 years from the date of first use.

The shelf life given is an indication. The following factors can cause it to vary greatly:

- Non-compliance with the manufacturer's instructions for transport, storage and use etc...
- Use of product other than it's intended use

Warning: these factors can cause damage and reduce the life of the product.

No modification or addition or repair should be carried out on the product without prior prior written consent from the manufacturer.

Do not use beyond the area of use defined in the instruction for use.

The manufacturer cannot be held liable for any direct or indirect accident after a modification or use other than that provided in this manual.

To ensure its working condition and therefore the safety of the user, the product should be checked systematically:

- **visual inspect the following points:**

- State of stretcher lifting sling hardware (buckle, D-rings, karabiners); these items must not be damaged, broken, distorted and must be free of sharp edges, burrs, cracks, worn parts, or corrosion.
- State of webbing: Material must be free of frayed, cut or broken fibers. Check for tears, abrasion, mold, burn or discoloration.
- Inspect stitches: check for pulled or cut stitches. Broken stitches may be an indication that the product has been impact loaded and must be removed from service.

PERIODIC REVIEW:

The examination must be performed at least every twelve months by the manufacturer or a competent organization, mandated by them. This very important check is related to the maintenance and efficacy of the product and hence the user's safety.

Replace the stretcher lifting sling if necessary.

The Record card must be completed prior to the first use of the product and then updated and kept with the product as well as the manual by the user,

The legibility of the product labeling should be checked periodically.

STORAGE INSTRUCTION :

During transport and storage :

-Keep the product in its packaging

-Keep the product away from any sharp, abrasive, object, etc...

-Keep the product away from sunlight, heat, flames, hot metal, oil, petroleum product, harsh chemicals, acids, dyes, solvents, sharp edges and structures of small diameter. These items can affect the performance of the product.

After use, store the product in its packaging in a cool, dry, ventilated area.

INSTRUCTION FOR CLEANING / MAINTENANCE :

Clean with soap and water with a cloth and hang in a ventilated area to dry naturally away from any direct fire or source of heat, even for items that got wet during use.

Bleach and detergents are strictly prohibited.

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